



Important – Confidentiality and Security information

Paternity Experts will **not** give out results or any information about your test over the phone unless a password is provided. Once this form is submitted, the contact information cannot be changed.

Collect and Send Contact Information Sheet (Please print clearly)

Contact Name: _____

Address: _____

City: _____ State: _____ Zip Code: _____

E-mail: _____

Relationship	Names of individuals tested	Cost	Alleged Father's Ethnicity (Race)
Alleged Father			<input type="checkbox"/> Caucasian <input type="checkbox"/> African <input type="checkbox"/> Asian <input type="checkbox"/> Hispanic <input type="checkbox"/> Native American <input type="checkbox"/> Other (Specify)
Child 1			
Child 2			
Mother			
Check one	<input type="checkbox"/> 5-Days <input type="checkbox"/> 3-Days <input type="checkbox"/> 2-Days		
Total Paid:			
How would you like the results reported to you? Check all that apply (You can choose all) <input type="checkbox"/> Mail (Discreet envelope) <input type="checkbox"/> E-mail <input type="checkbox"/> Will call for results (One call)			List any additional email addresses here:
Password			Anyone emailing or calling for information will be asked for a password. Provide password here

Please circle the method of payment: Credit Card Money Order

Pre-Pay Transaction Order ID #: _____ Date: _____ (From online order receipt)

If paying by credit card please fill in the appropriate information:

Name as it appears on the card: _____

Account Number: _____ Expiration Date: _____

Signature: _____ Date: _____ Amount authorized: _____

Credit Card users: billing address if different than above:

Address: _____ City: _____

State: _____ Zip Code: _____

Please complete this form and return it with your DNA samples



Call 541-688-4988 if you have any question or concerns or email us at info@paternityexperts.com

Collect and Send Collection Procedure:

You can use you ordinary Q-Tips or swabs to collect the DNA sample (the same swabs used to clean your ears). It is best to use an unopened or new pack of Q-Tips to prevent any contamination problems associated with frequently used swabs. We want the cells that line the surface of the cheek inside the mouth.

Step 1: We want two Q-tips or swabs for each person being tested. It is best to use one swab on the right side and the other swab on the left side of the inside of the cheek. **Do not touch the tip of the Q-tip.** Scrape and rotate the swab on the inside of the cheek, approximately, 10-20 seconds on each side. Repeat the process with the other swab on the other side of the mouth. For example if you used one swab on the left side use the other swab on the right side.

Step 2: Since a Q-Tip swab has two ends, cut the head of the swab off so that the part that was in the mouth falls off into a paper envelope (**DO NOT USE PLASTIC BAGS**). Both the left and the right swab can go into the same paper envelope. You do not need to label the envelope left or right the DNA is identical for both swaps, we just want to sources of DNA in case one fails.




Step 3: Label the paper envelope father or child depending upon the sample. Both the left and right swab from the same individual go into an envelope marked child and both the left and right swab go into separate paper envelope marked father.

Step 4: You will need two stamps to mail the envelope. Do not forget to include payment with your DNA sample along with the Collect and Send Submission form so we know how to contact you once the test is completed.

Send the Collect and send the DNA collection kit to:

For regular Mail Service use:
 Paternity Experts
 PO Box 40787
 Eugene OR 97404-0137

For FedEx or UPS, DHL and USPS next day use:
 Paternity Experts
 895 Country Club Rd, Ste C-250
 Eugene, OR 97401-6003

Collect and Send	Cost	Results*	
 Swab DNA Collection 2-Person Personal DNA Test Paternity or Maternity Testing	\$79	5-Days	5-day Test: If the samples are received on Monday results will be available on Monday.
	\$99	3-Days	3-day Test: If the samples are received on Monday results will be available on Thursday.
	\$139	2-Days	2-day Test: If the samples are received on Monday results will be available on Wednesday.
Additional Person	Cost	Person	
 Mother, child or father	\$70	Child	 Accredited by the BBB: Yes, Paternity Experts is accredited by the Better Business Bureau (BBB) to ensure that you are dealing with a reputable and reliable company. Ask Before You Send the Samples: If you have questions regarding the collect and send method, we encourage you to contact Paternity Experts. We prefer that you ask questions before you collect the samples to prevent any problems. Call 541-688-4988 if you have any questions.
	\$70	Father	
	\$70	Mother	

*Results are typically made available via email and phone between 12:00 p.m. to 2:00 p.m. (West Coast) on the day they are scheduled to be completed.